



RJ Cooper & Associates

Assistive Technology Specialist

1-800-RJCooper


Internet: <www.rjcooper.com> Email: <infoRJ@rjcooper.com>

Ultimate II Case (for iPad H-56 & iPad Mini H-91)

*NOTE: Instruction B under Insertion below is crucial.
I am not responsible for damage to that corner!*

Initial Removal of Black 'spacer' & Subsequent Removal of iPad/Mini

1) I use my Super-Velcro to keep the center of the bottom edge in place. The Super-Velcro strip on the Case is in place when you receive it. Please apply the mating Super-Velcro strip to the rear of your iPad/Mini as pictured →

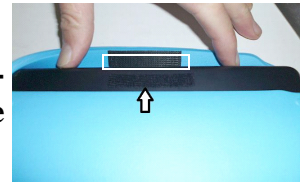

2) After you've applied the Super-Velcro, and you'll be removing your iPad/Mini from my Case, you will need to slide a credit card in between the 2 halves of my Super-Velcro. Keep the card between the Velcro's for steps 3 & 4!!
(you can choose to not use a card but you'll need to keep the 2 halves of the Super-Velcro apart as you're doing step 4)

3) Position the case and your hands/fingers like so. →




4) Push both your thumbs forward/upward to flip both corners over the iPad.

5) Now both corners are out of the way. Slide your iPad out. →



Insertion

A) With the 'lip' folded forward (toward the front of the Case), slide the iPad/Mini in all the way. →

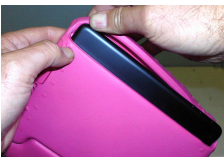

B) Critical!!! This is the step where you could ruin the case and I am not responsible! Flip THIS (upper left) corner over the iPad/Mini FIRST. The foam in that corner is the 'weakest link' of the entire case, so be careful!!!



C) While holding the Super-Velcro strip on the CASE up and away from the iPad/Mini, easily flip the upper right corner toward you, over the iPad/Mini's corner. →

If you are using a screen protector sheet, apply it before insertion.


Please read the back for handle removal/attachment and mounting!

The Handle

(Ultimate II Case for iPad/Mini)

The handle for my Ultimate II cases is removable. Even though most love it, because it folds back to work as a stand, of sorts, and it folds back when the iPad/Mini is used, on the go, while strapped around someone, there are times when it's desirable to remove the handle:

- Mounting it my Adjustable Tablet Holder (ATH) requires removal of the handle. (When ordering the ATH, for the iPad, use the Tall ATH. For the Mini, the Standard ATH works.)

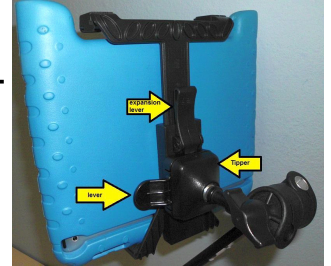


- If the handle is just too tempting for some with behavior issues, remove it!

Removal

1) Position and hold your iPad/Mini like this.



2) Hold tight to the upper left corner of the Case and pull down and to the right on the handle, so that the metal bar clears its hole.

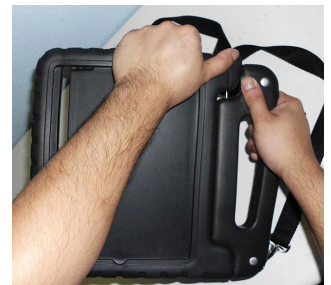

3) Remove the metal bar from the other side of the Case. The bar is glued to the handle. Try to 'clear' the foam with the bar so you don't damage the foam (we are NOT responsible for damage to the foam! :-)



Insertion



A) Insert the 'knobbed' end of the metal bar into the hole on the left of the case, as pictured here.

B) Pull up and to the left to set the other end of the metal bar into the Case. Try to 'clear' the foam with the bar so you don't damage the foam.
(We are NOT responsible for damage to the foam! :-)

